

DRUGCHECK[®]

Oral Fluid Test

Drug Screen Device

- Easy and convenient
- Eliminate controlled collection site
- Eliminate need for same-sex collectors
- Test can be performed anytime, anywhere
- No more collection fees



Forensic
use only



Step 1

Remove the collector from the sealed pouch. Remove the collection sponge from the collection chamber.



Step 2

Insert the sponge into the mouth. Close mouth and move the sponge around for oral fluid collection. Swab the inside of the mouth and tongue for 3 minutes until sponge is soft and fully saturated. The sponge will be free from hard spots when saturated.




Step 3

Place the saturated sponge into the collection chamber and turn the cap to secure it.



Step 4


Remove the dispenser cap.

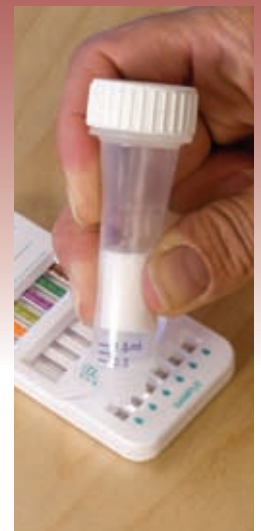

Step 5

Hold the collector upright and point the opening of the dispenser toward the specimen well of the test device. Use the thumb and index finger to squeeze the collection chamber and dispense one drop of collected oral fluid into each specimen well. Read results at 5 minutes.

Interpret Results

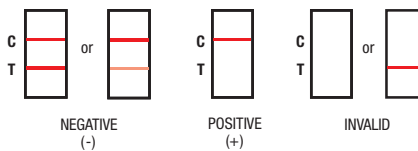

Test	Cut-off
Amphetamine (AMP)	50 ng/mL
Benzodiazepine (BZO)	50 ng/mL
Cocaine (COC)	20 ng/mL
Opiates (OPI)	40 ng/mL
Marijuana (THC)	12 ng/mL
Marijuana (THC)	200 ng/mL
Methamphetamine (mAMP)	50 ng/mL
Phencyclidine (PCP)	10 ng/mL

